

per VDE 0701-0702, IEC 62353 and IEC 60974-43

3-349-753-03 18/5.18

- 8 preconfigured test sequences for quickly testing simple operating equipment
- One universal, adjustable test sequence
- One test sequence executed with individual measurements
- Suitable for use by instructed persons
- Enormous data maintenance and storage concept for automated test sequences and measurements for up to 50,000 data records
- Fast access to mesurement and test functions with double rotary switch, direct selection keys and softkeys
- High-resolution, brilliant 4.3" TFT color display
- Unique multiple measurement allows convenient recording of several measuring points.
- Automatic DUT connection and protection class detection
- Compact, impact resistant housing with integrated rubber protector
- Comprehensive, legally secure preparation of test reports
- Modern interfaces: for data entry (two USB A) and data exchange (one USB B)
- Extensive setting options for international use (language, keyboard, character set, date, time)
- Testing of PRCDs of PRCD standard type, SPE-PRCD, PRCD-S and PRCD-K within test sequences in accordance with DIN VDE 0701-0702-PRCD.

Additional Functions SECUTEST PRO

- Remote control via PC software possible
- Additional database elements for property, building, floor, room for a better structuring of data and additional fields for department and cost center
- **Multi-print** read-out of all test reports which are available for a device under test with 1 finger tip (at a connected Z721S thermal printer)
- Design user-created report templates with "ReportDesigner" software (free available from myGMC)
- RFID transponder, read/write (Z751R,S,T), with SCANBASE RFID Z751E (UID or memory depending on how the reader is programmed)
- XML data export to a USB stick
- IZYTRONIQ or USB stick data import of all important data into the tester
- Design user-created sequences with "IZYTRONIQ" software

Added Feature SECULIFE ST BASE(25)

SECULIFE ST BASE corresponds to the range of functions offered by **SECUTEST PRO**, but is additionally endowed with antimicrobial properties. This is to curb the growth of germs, counteract microbial colonization or kill microorganisms.

SECULIFE ST BASE25 conforms to **SECULIFE ST BASE**, apart from being capable of testing with a 25 A instead of 10 A test current.



Standards for the Use of SECUTEST BASE/PRO and SECULIFE ST BASE(25) Test Instruments

	Testing after Repairs / Periodic Testing				
DUTs to be tested in accordance with the following standards	DIN VDE 0701-0702	IEC 62353 DIN EN 62353 (VDE 0751-1)	IEC 60974-4 DIN EN 60974-4 VDE 0544-4		
Electric devices: e. g. Work devices Mains operated electronic devices Hand-held electric tools Extension cords Household appliances Data processing devices	•				
Electrical medical devices		•			
Arc welding units	•		•		

Overview of Differences in Features

SECUTEST	Feature	BASE	PR0	PRO BT comfort	-
SECULIFE		—	ST BASE	—	ST BASE25
Touch screen / keyboard	E01		•	•	•
10 A RPE test current	G01		•	•	
25 A RPE test current	G02				•
2 nd test probe	H01		•	•	•
Voltage meas. inputs*	101		•	•	•
SECUTEST DB+	KB01		•	•	•
SECUTEST DB comfort	KD01			•	•
Bluetooth®	M01			•	
Antimicrobial housing	—		ST BASE		•

for voltage measurements or connecting current clamp sensors or AT3 adapter as well as for temperature measurement via RTD $\,$

Overview of Features Included with SECUTEST BASE, SECUTEST PRO and SECULIFE ST BASE(25) Test Instruments

Switch Set- ting		ing Function, rent/Voltage	Measurement Type Connection Type
Single	measure	ments, rotary switch level: green	
RPE	R _{PE}	Protective conductor resistance	PE(TS) - P1 passive
		Test current (200 mA)	PE(TS) - P1 active
	-	SECUTEST BASE10/PRO:	PE(Mains) - P1
		and SECULIFE ST BASE 10 A ¹ (Feature G01)	
		& SECULIFE ST BASE 25: 25 A ¹⁾ (Feature G02)	P1 - P2 ³
Riso	R _{ISO}	Insulation resistance	LN(TS) - PE(TS)
	U _{ISO}	Test voltage	LN(TS) - P1
			P1 - P2 ³ PE(Mains) - P1
			PE(INIAIIIS) - P1 PE(TS) - P1
			LN(TS) - P1//PE(TS)
IPE		Protective conductor current, RMS value	() ()
	I _{PE~}	AC component	Differential
		DC component	Alternative
	UIN	Test voltage	AT3-Adapter ²
-		Truck comment DMO control	Clamp ²
lв	I _T ∼	Touch current, RMS value	Direct Differential
	I _{T~}	AC component	Alternative (P1)
		DC component	Permanent connectio
	U _{LN}	Test voltage	Alternative (P1–P2)
lg	$I_{E\simeq}$	Device leakage current, RMS value	Direct
	I _{E~}	AC component	Differential
	$I_{E=}$	DC component	Alternative
	U _{LN}	Test voltage	AT3-Adapter ² Clamp ²
IA	$I_{A\simeq}$	Leakage current from the application part,	Direct (P1)
IA	'A≃	RMS value	Alternative (P1)
	U _A	Test voltage	Permanent conn. (P1)
IP		Patient leakage current, RMS value	
	I _{P~}	AC component	Direct (P1)
	I _{P=}	DC component	Permanent conn. (P1)
	U _{LN}	Test voltage	
U	U <u>~</u>	Probe voltage, RMS	PE - P1
	U	Alternating voltage component	PE - P1 (with mains*)
	U_	Direct voltage component	* polarity preset
		Measurement Voltage RMS ²	
	U	Alternating voltage component ²	V – COM
	U_	Direct voltage component ²	V – COM (with mains
ta 4	t _B	PRCD time to trip for 30 mA PRCDs	
		Line voltage at the test socket	
Р		n test at the test socket	
	I	Current between L and N	
	U	Voltage between L and N	
	f	Frequency	Polarity preset
	P	Active power	
	S	Apparent power	
	PF	Power factor	
Probe r		g functions	
EL1		cords with adapter:	EL1 adapter
		v_i short-circuit, polarity (wire reversal ⁵)	AT3-IIIE adapter VL2E adapter
EXTRA	Reserved	for expansion during the course of software	
	°C	Temperature measurement ² with Pt100/Pt1000	
	IZ	Measurement of current at clamp with	V – COM
	1	current clamp sensorn	

10 A/25 A-R_{PF} measurements are only possible with line voltages of 115/230 V and line frequencies of 50/60 Hz. 2

Voltage mesurement inputs only with SECUTEST PRO (or device with Feature I01) and SECULIFE ST BASE(25)

Terminal for 2nd test probe for 2-pole measurement only with SECUTEST PRO (or device with Feature H01) and SECULIFE ST BASE(25)

Measurement of time to trip not possible in IT systems 5

No checking for reversed polarity takes place when the EL1 adapter is used.

Key	
Alternative	= alternative measurement
	(equivalent leakage curre
Differential	= differential current meas

e current measurement) Differe nt measurement Direct = direct measurement LN(TS) = short-circuited conductors L and N of test socket P1 = measurement with test probe P1 P1-P2 = 2-pole measurement with test probe P1 & P2 PE-P1 = measurement between PE and test probe P1

PE(TS) = protective conductor of test socket

PE(Mains) = protective conductor of mains terminal

Switch Setting	Standard	Measurement Type, Connection Type
Automate	d test sequences, rol	tary switch level: orange
Preconfig	ured (freely configura	able) test sequences – Delivery Status
A1	VDE 0701-0702	Passive measuring method, test socket
A2	VDE 0701-0702	Active measurement type, test socket
A3	VDE 0701-0702-IT	Parameters configuration for EDP (active)
A4	IEC 62353 (VDE 0751)	Passive measurement type
A5	IEC 62353 (VDE 0751)	Active measurement type
A6	IEC 60974-4	Connection type: test socket
A7	IEC 60974-4	Connection type: AT16-DI/AT32-DI
A8	VDE 0701-0702	VDE 0701-0702, measurement type Extension Cord test (RPE, RISO), EL1/VL2E/AT3-IIIE adapter
AUT0	VDE 0701-0702	Active measurement type, test socket

Display with Selectable Language

The display panel consists of a backlit, color multi-display at which menus, setting options, measurement results, instructions and error messages, as well schematic and wiring diagrams appear.

The display and user prompting can be set to the desired language depending on the country in which the test instrument is used.

Data Entry

Data can be entered, for example, via a barcode reader connected to the USB port, a RFID scanner, a USB keyboard, or via the softkey keyboard when it appears at the display.

The touch screen of **SECUTEST PRO** (or devices with Feature E01) and SECULIFE ST BASE(25) allows for the convenient entry of data and comments while menu control is still based on softkeys.

Creating a Database

A complete test structure with data regarding customers, buildings*, floors*, rooms* and test objects can be created in the test instrument. This structure makes it possible to assign single measurements or test sequences to devices under test belonging to various customers. Manual single measurements can be grouped together into a so-called "manual sequence".

The SECUTEST PRO and SECULIFE ST BASE(25) test instruments and those instruments with database expansion (Feature KB01) enable the user to prepare a test structure by means of the IZYTRONIQ software at the PC for subsequent transmission to the test instrument.

only with SECUTEST PRO or with database expansion (Feature KB01) and SECULIFE ST BASE(25)

Data Interfaces

Structures set up in, and measurement data saved to the test instrument can be imported to **IZYTRONIQ** report generating software via the USB slave port. Data can then be archived at the PC, comments can be added with the software and reports can be generated.

The following input and output devices can be connected to the two integrated USB master ports:

- An external keyboard and a barcode or RFID reader,
- USB stick for data backup, import, export and reporting
- A printer

Software Update

The test instrument can always be kept current thanks to firmware which can be updated via the USB slave port.

Report Generating Functions

All of the values required for approval reports or device logbooks for electrical equipment (e.g. per ZVEH) can be measured with this instrument. The measured data can be documented and archived thanks to the measurement and test report that can be printed with a thermal printer connected to the USB port, or stored to a PC.

Automatic Detection of Measuring Point Changes

During protective conductor measurement, the test instrument recognizes whether or not the test probe is in contact with the protective conductor, which is indicated by means of two different acoustic signals. This function is very useful where several protective conductor connections need to be tested.

Mains Connection Analysis

Line voltage and frequency are measured and compared with the data specified in the setup menu. Momentary voltage or nominal voltage in accordance with the standard is required, for instance in order to extrapolate measured values for the leakage current measurement.

Automatic Detection of Mains Connection Errors

The device automatically recognizes mains connection errors if the conditions in the following table have been fulfilled. The user is informed of the type of error, and all measuring functions are disabled in the event of danger.

Type of Connection Error	Message	Condition	Measurements
Voltage at protective conductor PE to fin- ger contact (START / STOP key)	Display at the instrument	Press START /STOP button U > 25 V Button \rightarrow PE: < 1 M Ω^2	All measurements disabled
Protective conductor PE & phase conductor L reversed and/or neutral conductor N interrupted		Voltage at PE > 100 V	Impossible (no supply power)
Line voltage < 180 V / < 90 V (depending on mains)		U _{L-N} < 180 V U _{L-N} < 90 V	Possible under cer- tain circumstances ¹
Test on IT/TN system	Display at the instrument	$\begin{array}{c} \text{Connection} \\ \text{N} \rightarrow \text{PE} > 50 \text{ k}\Omega \end{array}$	Possible under cer- tain circumstances

10 A/25 A-R_{PE} measurements are only possible with line voltages of 115/230 V and line frequencies of 50/60 Hz.

if the test person is highly insulated, the following error message may appear: "Interference voltage at PE of mains connection"

Analysis of DUT Connection and Condition

Depending on the measurement or how the DUT is connected, the following states are checked and displayed before measurement is begun.

Control Function	Condition
Short-circuit test Short-circuit / starting curren	t R≤2,5Ω**
No short-circuit (AC test) R > 2,5 Ω **
Open-Circuit Voltage U_0 4.3 V, Short-Circuit Current I_K < 250 mA	
On test On (passive DUT) $R < 250 \text{ k}\Omega$
Off (active DUT) $R > 300 \text{ k}\Omega$
Open-Circuit Voltage U_0 230 V AC, Short-Circuit Current I_K < 1,5 mA	
Special test No prob	$R > 2 M\Omega$
Probe detected	d R < 500 k Ω
Protection class detection (only for country-specific (earth-contact)	plug variant)*
Protective conductor exists: PC	I R < 1 Ω
No protective conductor: PC	I R > 10 Ω
Safety shutdown	
Triggered at following residual current value (selectable)	> 10 mA / > 30 mA
Triggered at following residual current values (selectable)	
During leakage current measuremen	t > 10 mA
During protective conductor resistance meas	. > 250 mA
Connection test (only for country-specific (earth-contact) plug variation	ant)*
Checks whether the DUT is connected to the test socket.	
Power line of DUT exist	$R < 1 \Omega$
No power line of DU	$\Gamma R > 10 \Omega$
Insulation test	
DUT set up in a well-insulated fashion	n R≥500 kΩ
DUT set up in a poorly insulated fashion	n R < 500 k Ω
PELine – PETestsocket: Open-Circuit Voltage U_0 500 V DC, $I_{\rm K}$ < 2 r	nA
Overcurrent protection (shutdown)	
Shutdown in the event of a continuous flow of current via the test socket: Our test instruments SECUTEST BASE(10), PRO and SECULIFE ST BASE(25) allow for the active testing of devices with a nominal current (lo current) of up to 16 A. The test socket of the respective test instrument is equipped with 16 A fuses and the switching capacity of the internal relay also amounts to 16 A. Starting currents of up to 30 A are permissible. Fo devices under test which are expected to feature a starting current of mo than 30 A, we strongly recommend the application of a test adapter for higher starting currents: e.g. test adapter of the AT3 series	ad s s r

* applies to **M7050** with feature B00, B09 and B10

applies as from version 1.7.0;

previous condition \leq 1.5 Ω or > 1.5 $\Omega,$ respectively

Application

Regulations and standards in accordance with which the test instrument is manufactured and tested:


DIN EN 61010-1:2011 VDE 0411-1:2011	Safety requirements for electrical equipment for measurement, control and laboratory use – General requirements					
DIN VDE 0404, part 1: 2002	Test and measuring equipment for testing the electrical safety of electrical devices – General requirements					
DIN VDE 0404, part 2: 2002	 Equipment for testing after repairs and modifications, or periodic testing 					
DIN VDE 0404, part 3: 2005	 Equipment for periodic tests and tests prior to commission- ing medical electrical devices or systems 					
DIN EN 60529/ VDE 0470, part 1	Test instruments and test procedures Degrees of protection provided by enclosures (IP code)					
DIN EN 61326-1 VDE 0843-20-1	Electrical equipment for measurement, control and laboratory use – EMC requirements – Part 1: General requirements					
IEC 61557-16	Electrical safety in distribution systems up to 1000 V a.c and 1500 V d.c – Equipment for testing, measuring or monitoring of protective measures - Part 16: Equipment for testing the safety of electrical equipment and medical electrical equipment according to IEC 62638 and IEC 62353 (IEC 85/437/CD:2012)					

Backlit Multi-Display Samples

Single Test – Initial Screen with Parameters Display



Help – Schematic and Wiring Diagram

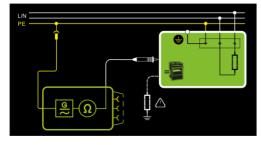

Test Function for Test Step in the Test Sequence

Fun	ction				ţ	
	omment entered!					
	0.01	А	Р	2	W	
U	228.9	۷	S	2	VA	ľ
f	50.0	Ηz	PF	1.00		X
_						
	\checkmark		м	anual rating		\mathbf{Z}

Results of a Test Sequence per VDE 0701-0702

Test						\bigtriangleup
VDE0701-0702		24/09/	2013 01:5	8:24 p	om 🗸	\bigtriangledown
DUT passed! ShortedCheck L-N					~	Æ.
Vis. Insp. RPE	≤300	mΩ	5	mΩ	~	
RINS PC I	≤300 ≥1.00	MΩ	ر 300 <	MΩ	✓ ✓	
IPE LN	≤3.50 ▽	mA	5	μA	~	\checkmark

Database Structure - List of Test Results



Scope of Delivery

Standard version (country-specific)

- 1 SECUTEST BASE, SECUTEST PRO or SECULIFE ST BASE(25) test instrument
- 1 Mains power cable
- 1 Test probe, 2 m, not coiled
- 1 USB cable, USB A to USB B, 1.0 m long
- 1 Plug-on alligator clip
- 1 KS17-ONE cable set for voltage measuring inputs (only with SECUTEST PR0 or devices with Feature I01) and SECULIFE ST BASE(25)
- 1 Calibration certificate
- 1 Condensed operating instructions D, GB
- Detailed operating instructions available on the Internet at www.gossenmetrawatt.com
- 1 Card with registration key for software



List of Software Variants depending on Device Type

IZYTRON JQ		Soft Varia	ware ants	
	Article number	BUSINESS Starter	BUSINESS Advanced	BUSINESS Professional
Standard Models				
SECUTEST BASE	M705A	•		
SECUTEST PRO	M705C	•		
SECUTEST PRO BT comfort	M705E	•		
SECULIFE ST BASE	M693A	•		
SECULIFE ST BASE 25	M693B	•		
Device Sets				
STARTER PACKAGE SECUTEST BASE	M706A		•	
MASTER PACKAGE DB+	M706D			•
PROFI PACKAGE SECUTEST PRO	M706M			•
WELDING PACKAGE SECUTEST PRO	M706P			•
3-PHASE CURRENT PACKAGE Secutest pro	M706S			•

IZYTRONIQ is a test software that has been newly developed from scratch. It enables the user to visualize and manage the entire testing procedure for all our test instruments and to document it in an audit-proof manner. For the first time, it is thus possible to combine the test and measurement data from a great variety of test instruments and multimeters in one test and generate one report report thereof. The intuitive user guidance and modern design provide for quick access to all functions.

The software is available in different sizes and versions for trades, industry and vocational training purposes.

Characteristic Values

Func-	Measured	Display Range / Nominal Range of	Reso-	Nominal Voltage	Open- Circuit	Nom. Current	Short- Circuit	Inter- nal Resis-	Refer- ence Resis-	Measuring	Intrinsic Error ¹		rload acity
tion	Quantity	Quantity Use	lution	U _N	Voltage U ₀	I _N	Current I _K	tance R _l	tance R _{REF}	Uncertainty ¹		Value	Time
	Protective conductor	1 333 11152	1 mΩ		< 24 V		>200 mA AC or DC > 10 A AC 5			±(15% rdg. + 10 D) > 10 D	±(10% rdg.+ 10 d)	264 V 250 mA	Cont.
	resistance	4.00 000 0	10.0		AC or DC		>35 AAC 11)			$>$ 10.0 Ω :	> 10 d	16 A ⁵	
751)	Rpe	1.00 999 Ω	10 mΩ				11)			±(10% rdg.+ 10 d)		>42 AAC 11)	15 s
0.0		10.0 … 30.0 Ω 10 … 999 kΩ	100 mΩ 1 kΩ										
2	Insulation	1.00 9.99 MΩ	10 kΩ	50 500						$\pm (5\% \text{ rdg.} + 4 \text{ d})$ > 10 d	±(2.5% rdg.+2 d) > 10 d		
353	resistance ⁹	10.0 99.9 MΩ	100 kΩ	50 500 V DC	1.0 ● U _N 1.5 ● U _N	> 1 mA	> 2 mA	—	—			264 V	Cont.
62	Riso	100 300 MΩ	1 MΩ							\geq 20 MΩ: ±(10% rdg.+ 8 d)	≥ 20 MΩ: ±(5% rdg.+4 d)		
Ē		0.0 99 μA	1 μA							((****3**/		
12)/	Leakage current, alternative	100 999 µA	1 μA		50	1 kΩ	\pm (5% rdg.+ 4 d) > 10 d	\pm (2% rdg.+2 d) > 10 d					
010	measurement ²	1.00 9.99 mA	10 µA		250 V~	—	> 1.5 mA	$>$ 150 k Ω	±10 Ω	> 15 mA:	> 15 mA:	264 V	Cont.
201	IPE, IB, IG, IA	10.0 30.0 mA	100 µA		- 20/+10%					±(10% rdg.+ 8 d)	±(5% rdg.+ 4 d)		
Tests, 62638 (DIN VDE 0701-0702) / IEC 62353 (VDE 0751)		Only lp: 0.0 99.9 μA	100 nA										
NO	Leakage current, direct	0.0 99 μA	1 µA					1 kΩ		±(5% rdg.+ 4 d)	±(2.5% rdg.+2 d)		
38 (measurement ³	100 999 µA	1 μA		—	_	—	±10 Ω		±(5% lug.+ 4 u) > 10 d	±(2.3% rug.+2 u) > 10 d	264 V	Cont.
326;	IPE, IB, IG, IA, IP	1.00 9.99 mA	10 µA										
ts, 0		10.0 30.0 mA	100 µA										
Tes	Leakage current,	0 99 µA	1 µA										
	differential	100 999 μA	1 µA	-		_		_		\pm (5% rdg.+ 4 d) > 10 d	\pm (2.5% rdg.+2 d) > 10 d	264 V	
	current measurement ⁴	1.00 9.99 mA	10 µA		—								Cont.
	IPE, IB, IG	10.0 30.0 mA	100 µA										
ket	Line voltage U _{L-N} ¹⁰	100.0 240.0 V~	0.1 V		_					_	±(2% rdg.+2 d)	264 V	Cont.
soc	Load current I	0 16.00 A _{RMS}	10 mA	_	_	_	_	_	_		±(2% rdg.+2 d)	16 A	Cont.
test	Active power P	0 3700 W	1 W		_	_	_	_	_	_	±(5% rdg.+10 d)	264 V	Cont.
st at		0 3700 W	1 44								> 20 d	20 A	10 min
Function test at test socket	Apparent power S	0 4000 VA	1 VA			Calo	culated valu	e, U _{L−N} • I _V	1		\pm (5% rdg.+10 d) > 20 d		
Func	Power factor PF with sinusoidal waveform: cosφ	0.00 1.00	0.01			Calculated	l value, P / S	S, display >	• 10 W		±(10% rdg.+5 d)		
t _a Prcd	Time to trip	0.1 999 ms	0.1 ms	_	_	30 mA	—	_	_	±5 ms			
	Probe voltage (test probe P1 to PE) $$, \sim and $=$	0,0 99.9 V	100 mV					3 MΩ			±(2 % v.M.+2 D)	300 V	
×	Measurem. voltage (sockets V–COM ⁶) , ~ and ऌ	100 250 V	1 V				_	1 MΩ			$\begin{array}{l} \pm (2 \ \% \ rdg. +2 \ d) \\ > 45 \ Hz \ \ 65 \ Hz \\ \pm (2 \ \% \ rdg. +5 \ d) \\ > 65 \ Hz \ \ 10 \ kHz \\ \pm (5 \ \% \ rdg. +5 \ d) \\ > 10 \ kHz \ \ 20 \ kHz \end{array}$	and ≂	Cont.
V-COM	Leakage current	0,00 0.99 mA ~	0.01 mA								±(2 % rdg.+2 d)		
>	via AT3-IIIE	1,0 9.9 mA ~	0.1 mA	_	_	_		_	_	_	$\pm (2\% 109.+20)$ > 10 D	253 V	Cont.
	adapter Z745S ⁶ ⁸	10 20 mA ~	1 mA								without adapter		
	Temperature with Pt100 sensor	– 200.0 + 850.0 °C											
	Temperature with Pt1000 sensor	– 150.0 + 850.0 °C	0.1 °C		< 20 V -		1.1 mA			_	±(2 % rdg.+1 °C)	10 V	Cont.

Func-	Measured	Display Range / Nominal Range of	Reso-	Nominal Voltage	Open- Circuit	Nom. Current	Short- Circuit	Inter- nal Resis-	Refer- ence Resis-	Measuring	Intrinsic Error ¹		rload acity
tion	Quantity	Use	lution	U _N	Voltage U ₀	I _N	Current I _K	tance R _I	tance R _{REF}	Uncertainty ¹		Value	Time
	Current via	1 99 mA \sim	1 mA (1 mV)										
	current clamp sensor	0.1 0.99 A \sim	0.01 A (10 mV)	_	_	_	_	_	_	_			
	[1 mV : 1 mA] (V–COM sockets ⁶ ⁷)	1.0 9.9 A ~	0.1 A (100 mV)										
		10 300 A \sim	1 A (1 V)										
	Quirrent uie	0.1 9.9 mA \sim	0.1 mA (1 mV)										
	Current via current clamp	10 99 mA ~	1 mA (10 mV)										
	sensor [10 mV : 1 mA] (V–COM sockets ^{6 7})	0.10 0.99 A ~	0.01 A (100 mV)	_	_		_	_	_	_			
		1.0 30.0 A ~	0.1 A (1 V)								\pm (2 % rdg.+2 d) > 10 d		
I _{Clamp}	Quirrent uis	0.01 0.99 mA \sim	0.01 mA (1 mV)								20 Hz 20 kHz without clamp	253 V	Cont.
	Current via current clamp sensor	1.0 9.9 mA \sim	0.1 mA (10 mV)										
	[100 mV : 1 mA] (V–COM sockets ^{6 7})	10 99 mA \sim	1 mA (100 mV)										
		0.10 3.00 A \sim	0.01 A (1 V)										
	Quarteraturia	1 99 µA ~	1 μΑ (1 mV)										
	Current via current clamp	0.10 0.99 mA \sim	0.01 mA (10 mV)										
	sensor [1000 mV : 1 mA] (V–COM sockets ^{6 7})	1.0 9.9 mA \sim	0.1 mA (100 mV)	_	_	_	_	_	_	—			
		10 300 mA \sim	1 mA (1 V)										

Specified values are only valid for the display at the test instrument. Data transmitted via the USB port may deviate from these values.

- 2 Known as equivalent leakage current or equivalent patient leakage current from previous standards 3
- Protective conductor current, touch current, device leakage current, patient leakage current 4
- Protective conductor current, touch current, device leakage current 5
- Only with SECUTEST BASE10 (Feature AA02), SECUTEST PRO and SECULIFE ST BASE 6
- Only with SECUTEST PRO (Feature IO1) and SECULIFE ST BASE
- 8
- Measurement type IPE clamp and IG clamp Measurement type IPE AT3 adapter and IG AT3 adapter 9
- The measuring range upper limit depends on the selected test voltage. ¹⁰ Due to inrush current limiting components, the voltage at the test socket may be lower than the measured line voltage.
- ¹¹⁾only with SECULIFE ST BASE25

Key: rdg. = reading (measured value), d = digit(s)

Test Times, Automated Sequence

The test times (parameter "Measurement duration ...") can be adjusted in the sequence parameter setting menu for each rotary switch position separately. The test times are not tested and calibrated.

Emergency Shutdown During Leakage Current Measurement

As of 10 mA of differential current (can also be set to 30 mA), automatic shutdown ensues within 100 ms. This shutdown is not effected during leakage current measurement with clamp or adapter.

Influencing Quantities and Influence Error

Influencing Quantity / Sphere of Influence	Designation per IEC 61557-16	Influence Error $\pm \dots \%$ rdg.
Change of position	E1	—
Change to test equipment supply voltage	E2	2.5
Temperature fluctuation	E3	Specified influence error valid starting with temperature changes as of 10 K:
0 40 °C		2.5
Amount of current at DUT	E4	2.5
Low frequency magnetic fields	E5	2.5
DUT impedance	E6	2.5
Capacitance during insulation mea- surement	E7	2.5
Waveform of measured current		
49 51 Hz	E8	2 with capacitive load (for equiva- lent leakage current)
45 100 Hz		1 (for touch current)
		2.5 for all other measuring ranges

Reference Ranges

Line voltage Line frequency Waveform Ambient temperature +23 °C ±2 K Relative humidity Load resistance

230 V AC ±0.2% 50 Hz ±2 Hz

Sine (deviation between effective and rectified value < 0.5%) 40 ... 60% Linear

Nominal Ranges of Use

Nominal line voltage 100 V ... 240 V AC Nominal line frequency50 Hz ... 400 Hz Line voltage waveform Sinusoidal 0 °C ... + 40 °C Temperature

Ambient Conditions

Storage temperature - 20 °C ... + 60 °C Relative humidity Max. 75%, no condensation allowed Elevation Max. 2000 m Deployment Indoors, except within specified ambient conditions

Power Supply

Electrical system Line voltage Line frequency Power consumption TN. TT or IT 100 V ... 240 V AC 50 Hz ... 400 Hz 200 mA test: approx. 32 VA

Mains to test socket (e.g. function test)

10 A test: approx. 105 VA

Continuous max. 3600 VA, power is conducted through the instrument only, switching capacity ≤ 16 A, ohmic load; for currents > 16 A AC please use the adapter AT3-IIS32 (Z745X)

Electrical Safety

Protection class	l per IEC 61010-1/EN	61010-1/VDE 0411-1				
Nominal voltage	230 V					
Test voltage	2.3 kV AC 50 Hz or 3	.3 kV DC				
	(mains circuit / test soc nal, USB, finger contac					
Measuring category	250 V CAT II					
Pollution degree	2					
Safety shutdown	At DUT differential current of > 10 mA,					
	shutdown time: < 100 ms,					
	can also be set to > 3					
	with following probe current during:					
	- Leakage current meas.:					
	> 10 mA~/< 5 ms					
	 Protective conductor > 250 mA~/< 1 ms 					
	At continuous flow of	current I > 16,5 A				
Fuse links	Mains fuses:	2 ea. FF 500V/16A				
	Probe fuse:	M 250V/250mA				
	SECUTEST BASE10/PR0	/				
	SECULIFE ST BASE:					
	Additionally (Feature G0	1) 1 ea. FF 500V/16A				

Electromagnetic Compatibility

Product standard DIN EN 61326-1

i louuoi siulluulu		
Interference Emission		Class
EN 55011		В
Interference immunity	Test value	Evaluation criterion
EN 61000-4-2	Contact/atmos. – 4 kV/8 kV	A
EN 61000-4-3	3 V/m or 1 V/m	A
EN 61000-4-4	1 kV	В
EN 61000-4-5	1 kV or 2 kV	A
EN 61000-4-6	3 V/m	A
EN 61000-4-11	0.5/1/25 periods	A
	250 periods	С

USB Data Interface

Туре	
Туре	

USB slave for PC connection 2 ea. USB master for data input devices* with HID-Boot interface, for USB stick for data backup, for USB stick for storing reports as bmp files, for printer*

compatible devices see next page

As of firmware version 1.6.0: In the remote operating mode, the test instrument can be controlled via the USB slave data interface. Pertinent interface commands are available upon request.

$Bluetooth^{ entropymber 8 2.1 + EDR Data Interface$ (SECUTEST PRO BT comfort only or feature M01) for remote control

Туре

Mechanical Design

Display	4.3" color display (9.7 x 5.5 cm), backlit, 480 x 272 pixels at 24 bit color depth, (true color)
Touch screen	with SECUTEST PRO/SECULIFE ST BASE(25) or feature E01
	(touch-sensitive user interface)
Dimensions	W x H x D: 295 x 145 x 150 mm
	Height with handle: 170 mm
Weight	SECUTEST BASE(10)/PR0: Approx. 2.5 kg SECULIFE ST BASE25: approx. 4.0 kg
Protection	Housing: IP 40
	Test socket: IP 20 per DIN VDE 0470, part 1/EN 60529,
	SECULIFE ST BASE(25): Housing with antimicro- bial properties in accordance with the JIS- Standard Z 2801:2000

Accessories (not included)

Z751A Barcode Reader

For connection to the USB master port at the test instrument, and for reading in barcodes. This makes it possible to conveniently insert the ID numbers of DUTs into single measurements and test sequences.

This device is based upon the concept of an instinctive scanning distance and provides best possible reading performance. Green Spot technology provides a "good-read" projection directly on the code. The device is equipped with a USB port.

Barcode printer Z721E

For connection to the USB master port at the test instrument, and for printing out barcode labels.

Codina: Code39, Code128, EAN13, Text, QR Code*, Micro QR Code, DataMatrix, Aztec

QR Code is a registered trademark of DENSO WAVE INCORPORATED

Z721S Thermal Printer

For connection to the USB master port at the test instrument, and for printing out test reports.





The Z745A CEE adapter allows for quick and efficient testing of devices equipped with a CEE plug. The adapter is equipped with the following CEE flush-type socket outlets: 5-pole 16 A, 5-pole 32 A and 3-pole 16 A. Furthermore, the adapter includes five 4 mm safety sockets to which 3-phase devices without permanently attached plug or conventional measurement cables can be connected, e.g. by means of quick clamp terminals (not included). The following tests can be performed on devices with CEE plugs with the help of the adapter:

- Testing of protective conductor continuity
- Insulation resistance, alternatively leakage current (equivalent leakage current)
- Function test (3-pole CEE outlet only)

The Z745A CEE adapter may also be used as an adapter for connecting devices with 3-pole CEE plugs to common earthing contact outlets.

VL2 E (Z745W)

Test adapter with single-phase and 3-phase plug connectors up to CEE 32A



AT16-DI (Z750A) 3-Phase 16 A Differential Current Adapter

Devices which are equipped with a 5pole, 16 A / 6 h CEE plug can be quickly and efficiently tested with the AT16-DI CEE adapter. The following tests can be performed on devices with



Testing of protective conductor continuity

help of the AT16-DI

CEE adapter:

- Insulation resistance, alternatively leakage current (equivalent leakage current)
- Measurement of protective conductor resistance with the following methods: equivalent leakage current / differential current / direct
- Function test

This differential current adapter is also available in a variant with a 5-pole 32 A / 6 h CEE plug with the designation AT32-DI CEE adapter.

SCANBASE RFID (Z751E) (RFID read / write)

Compact write/read device with USB interface for programming and reading of 13.56 MHz transponders per ISO 15693.

SECUTEST BASE10/PR0/SEC-ULIFE ST BASE(25) enable the user to populate the RFID tags directly from the test instrument with the help of the programmer.



SECU-cal 10 (Z715A) Calibration Adapter

The calibration adapter is used for testing the measuring uncertainty of test instruments in accordance with DIN VDE 0701-0702 / IEC 62353 (VDE 0751). As a rule, these instruments must be tested once each year, as well as for certifi-



cation in accordance with the ISO 9000 quality standard, as set forth by accident prevention regulation DGUV provision 3 (previously BGV A3).

All limit values for the required tests per DIN VDE, as well as protective conductor resistance, insulation resistance, equivalent leakage current, differential and/or touch as well as housing leakage current, must be tested.

SECULOAD-N (Z745R) Test Adapter

Test Adapter for testing open-circuit voltage at welding units per IEC/ EN 60974.

In combination with the test instrument, the test adapter is used for testing welding units in accor-



dance with the IEC/EN 60974-4 standard. This standard stipulates that peak values for open-circuit voltage may not exceed the limit values, regardless of the utilized settings.

SECUTEST BASE(10)/PRO/SECULIFE ST BASE(25) testing instrument includes a test sequence for testing welding instruments with this adapter.

The peak value rectifier of the SECULOAD-N uses rectifier diode 1N 4007 recommended by the standard. This diode is a power rectifier diode and, due to its design principle, only suitable for voltage sources with a low clock rate in the line frequency range or for voltage sources with conventional transformers.

EL1 (Z723A) Adapter for Testing Single-Phase Extension Cables



AT3-III-E (Z745S) 3-phase Current Adapter

Test adapter for active and passive testing of Single and 3-Phase Electric Devices and Extension Cables in Combination with SECUTEST... Test Instruments

Operation is simple and safe. The test adapter is connected to a 3-phase 16 A mains outlet, and to the respective test instrument. Testing is performed without reversing polarity at the



device under test, either automatically or manually, and is controlled by the test sequence of the utilized test instrument. Safety shutdown occurs if the factory preset residual current value is exceeded.

SORTIMO L-BOXX (Z503D)

Plastic system case Outside dimensions: W x H x D 450 x 255 x 355 mm Foam insert Z701D for tester and accessories, has to be ordered seperately, see below.



Foam insert for SORTIMO L-BOXX (Z701D)



F2000 Universal Carrying Pouch (Z700D)

Test instrument, plug inserts, measuring adapters, replacement batteries, recording charts etc. can be stored in a clear-cut fashion and conveniently transported in the F2000 carrying pouch. Outside dimensions: 380 x 310 x 200 mm (without buckles, handle and carrying strap)



Order Information

SECUTEST BASE, SECUTEST PRO, SECULIFE ST BASE and SECULIFE ST BASE 25 Standard Models

Standard Model	Article Number	Features
SECUTEST BASE	M705A	Schuko variant (test socket and mains plug), selectable user interface language (default setting: German), protective conductor test current: 200 mA, calibration certificate in D/GB/F, printed condensed operating instructions in German (features differing from 00: AA01 V01) including IZYTRON.IQ BUSINESS STARTER
SECUTEST PRO	M705C	same design as M705A, additionally with 10 A RPE test current, with touch screen, voltage measuring inputs, sockets für 2 nd test probe and database expansion DB+ (features differing from 00: AA03 E01 G01 H01 I01 KB01 V01) including IZYTRON.IQ BUSINESS STARTER
SECUTEST PRO BT comfort	M705E	same design as M705C, additionally with Bluetooth interface and database comfort (features differing from 00: AA03 E01 G01 H01 I01 KB01 KD01 M01 V01) including IZYTRON.IQ BUSINESS STARTER
SECULIFE ST BASE	M693A	same design as M705C, additionally with antimicrobial housing (features differing from 00: A01 AA11 E01 G01 H01 I01 KB01 V01) including IZYTRON.IQ BUSINESS STARTER
SECULIFE ST BASE 25	M693B	same design as M693A, however with 25 A RPE test current (features differing from 00: A01 AA12 E01 G02 H01 I01 KB01 KD01 V01) including IZYTRON.IQ BUSINESS STARTER

Order Information on Device Kits

Туре	Designation						Article Number
Starter Package SECUTEST BASE	Lieferumfang siehe unten including IZYTRON.IQ BUSINESS ADVANCED						M706A
Master Package DB+	Lieferumfang siehe unten including IZYTRON.IQ BUSINESS PROFESSIONAL						M706D
Profi Package SECUTEST PRO	Lieferumfang siehe unten including IZYTRON.IQ BUSINESS PROFESSIONAL						M706M
Welding Package SECUTEST PRO	Lieferumfang siehe unten including IZYTRON.IQ BUSINESS PROFESSIONAL						M706P
3-PHASE CURRENT PACKAGE SECUTEST PRO	Lieferumfang siehe unten including IZYTRON.IQ BUSINESS PROFESSIONAL						M706S
Accessories	For use in combination with the following testing packages:	Starter Package	Master Pack. DB+	Profi Package	Welding Package	3-PHASE CURRENT Package	
SECUTEST BASE							M705A
SECUTEST BASE10*							AA02, G01, KB01
SECUTEST PRO				Z03 🔳	Z04 🔳	Z05 🔳	M705C
SORTIMO L-BOXX	Plastic system case				2 x 🔳		Z503D
Foam SORTIMO L-BOXX Secutest4	Foam insert for SORTIMO L-BOXX with compartment for SECUTEST BASE(10) or PRO						Z701D
FOAM SORTIMO L- BOXX-Adapter	Foam insert for SORTIMO L-BOXX with compartment for adapter						Z701E
EL1	Adapter for the testing of single-phase extension cables						Z723A
Brush Probe	Contact brush						Z745G
SECULOAD-N	Test adapter in combination with SECUTEST for testing welding units per DIN EN 60974-4:2007.						Z745R
Adapter AT16-DI	3-Phase 16 A Current Adapter with Residual Current Logging						Z750A
SK2	Probe cable with test probe and 2 m probe cable (not coiled)						Z745D
SK5	Probe with probe tip and 5 m probe cable (not coiled) for protective conductor measurement,						Z7450
Adapter cable CEE16/CEE32	Adapter cable CEE 16 A to CEE 32 A						Z750F
Barcode scanner	Barcode scanner for USB connection						Z751A
Thermal printer	Thermal printer for printing out test reports; including manual on CD, Lith- ium battery, power supply adapter, mains cable, 1 role of thermal paper						Z721S
		Kev: II in	cluded 🖵 (optional		1	1

* Database expansion DB+ included

. . ..

Order Information for Accessories	Tuno	Article number	Designation Calibration adapter	Туре	Article number
Designation	Туре	Article number	Calibration adapter		
			DIN VDE 0701-0702/IEC 62353		
Mains power cable Cable set for connecting test instruments			(VDE 0751) (max. 200 mA) cannot be		
to the mains without using a an earthing			used for 10 A protective conductor test		
contact outlet, and for connecting DUTs.			current	SECU-cal 10	Z715A
Consists of coupling socket with 3 perma-					
nently connected cables, 3 measurement			Probe cable		
cables, 3 plug-on pick-up clips and 2 plug-			Probe with probe tip and 2 m probe cable		
on test probes.	KS13	GTY3624065P01	(not coiled), 25 A short-circuit operation:		
			"on-time" 60 s, "resting time" 120 s		
Adapter for testing 3-phase current con	sumers		for SECULIFE ST BASE 25	SK2-25A	Z746C
Adapter for connecting DUTs:			Probe cable with test probe and 2 m probe	01/0	77 450
3-pole 16 A, 5-pole 16 A + 32 A,			cable (not coiled), 300 V CAT II 16 A	SK2	Z745D
5 ea. 4 mm socket			Probe cable with test probe and 2 m probe	01/01/1	77451
- For all tests without line voltage			cable (coiled), 300 V CAT II 16 A	SK2W	Z745N
at single and 3-phase electrical devices - for differential current measurements			5 m probe cable for protective conductor measurement, 300 V CAT II 16 A	SK5	Z7450
(direct or differential current method)	CEE Adapter	Z745A	Brush probe	Z745G	Z745G
6 A / 32 A 3-phase current adapter (test case)				27430	27430
- For all tests without line voltage at single			Multiple probe connector for connecting 5 • 4 mm and 5 • 2 mm test probes to mea-		
and 3-phase electrical devices			sure multiple touchable housing parts or		
- For tests at single			application parts.	SV5	Z745J
and 3-phase extension cords			Cable set (1 pair of measuring cables) 1.2 m,	-	
 For differential current measurements 			with VDE-GS sign 1000 V/CAT III 1 A,		
(direct method)			600 V/CAT IV 1 A,		
- für leakage current measurements in			1000 V/CAT II 16 A*	KS17-2	GTY3620034P0002
accordance with differential current method ¹	AT3-III-E ^D	Z745S	2 each in plastic bag, diameter 4 mm, length		
	AI 3-III-E	27400	1.0 m, 1000 V CAT III, 19 A, blue	Cable set blue	Z746A
Test adapter for tests on devices with CEE16 and CEE32 connections			2 each in plastic bag, diameter 4 mm, length	.	
load rating of max 20 A)	AT3-IIS D 1	Z745T	1.0 m, 1000 V CAT III, 19 A, black/red	Cable set bw/rd	Z746B
same as AT3-II-S, however, with a load	710 110	21101			
ating of 32 A	AT3-II S32 ^{D 1}	Z745X	Clip-on current sensor for SECUTEST PR	0/Seculife ST BA	SE(25)
B-phase 16 A differential current adapter	AT16-DI	Z750A	Clip-on current sensor, can be set to		
B-phase 32 A differential current adapter	AT32-DI	Z750B	1 mA to 15 A or 1 A to 150 A,		
Test adapter with single and 3-phase plug			frequency range: <u>45 65</u> 500 Hz, 1 mV/mA and 1 mV/A	WZ12C D)	Z219C
connectors up to CEE 32A			Leakage current clamp 0.1 mA 25 mA,	WZ120	22130
- For all tests without line voltage at single			100 mV/mA	SECUTEST CLIP D)	Z745H
and 3-phase electrical devices					
- For tests at single		77 45141	Temperature sensors for SECUTEST PRO	/SECULIFE ST BAS	E(25)
and 3-phase extension cords	VL2E	Z745W	Pt100 temperature sensor for surface and		
Adapter cable CEE 16 A 5-pin plug red on CEE 32 A 5-pin coupling red, 0.5 m,	Adaptar appla		immersion measurement, -40 to + 500 °C	Z3409	GTZ3409000R0001
$5 \times 1.5 \text{ mm}^2$	Adapter cable CEE16/CEE32	Z750F	Pt1000 temperature sensor for measure-		
5 × 1.5 mm	ULL TO/ULLJZ	27,501	ment in gases and liquids,		
Adapter for testing single-phase extensi	on cables		−50 +220 °C	TF220	Z102A
Adapter for testing single-phase extension			Pt100 oven sensor,		0770 (
cables including earth contact and inlet			Pt100, -50 +550 °C	TF550	GTZ3408000R0001
olug inserts	EL1	Z723A	Sounding pipe oil temperature sensor,		
Plug insert for using adapter EL1			Pt1000 class B, -50+500 °C, sensor 3 mm dia. x 810 mm length	TEADOCAR	Z102C
n Switzerland	PRO-CH	GTZ3225000R0001		TF400CAR	21020
Adapter for testing welding units					
Fest adapter in combination with					
SECUTEST for testing welding units per					
DIN EN 60974-4:2007.					
The peak-value rectifier in the SECULOAD-					
N uses the 1N4007 rectifier diode recom- nended in the standard.					
This is a mains rectifier diode which, due to					
ts design, is only suitable for voltage					
sources with low cycle rates within the					
ange of the line frequency, or voltage					
sources with conventional transformer.					
ncludes 4 measurement cables and 2 alli-					
gator clips.	SECULOAD-N	Z745R			

Designation	Туре	Article number
Pouches and Cases		
Carrying pouch for SECUTEST BASE(10)/	_	
PRO/SECULIFE ST BASE	F2000 ^D	Z700D
Carrying pouch big for tester sets	F2020	Z700F
Universal carrying pouch with flexible di-		
vider and display protection for SECUTEST		
BASE(10)/PRO/SECULIFE ST BASE	F2010	Z700G
Plastic system case	SORTIMO L-BOXX	Z503D
Foam insert for SORTIMO L-BOXX with di-		
vider for SECUTEST BASE(10)/PRO/SECU-	Foam SORTIMO	
LIFE ST BASE	L-BOXX Secutest4	Z701D
Foam insert for SORTIMO L-BOXX GM with	Foam SORTIMO	
divider for adapters	L-BOXX Adapter	Z701E
Data Storage		
Database expansion for SECUTEST		
BASE(10): data import, sequence import,		
Remote	SECUTEST DB+	Z853R
Database extension "comfort" for		
SECUTEST BASE(10)/PRO/SECULIFE ST		
BASE(25)		
Entry option for test interval and medical		
device, shifting of test objects, Touch Edit,		
Quick Edit, sending of test result (to inter-		
face), Autostore		
Please indicate the SECUTEST serial num-	SECUTEST DB	
	comfort	Z853S

Designation	Туре	Article number
Report Generating Accessories		
RFID-System		
RFID read/write for USB connection		
(frequency: 13.56 MHz)	SCANBASE RFID	Z751E
RFID tags per ISO 15693, dia. approx.		
22 mm, self-adhesive, 500 pcs.	Z751R	Z751R
RFID tags per ISO 15693, dia. approx.		
30 mm, thickness 2 – 3 mm with 3 –		
4 mm hole 500 pcs.	Z751S	Z751S
RFID tags per ISO 15693, pigeon ring,		
dia. approx. 7.5 mm, 250 pcs.	Z751T	Z751T
Barcode reader		
Barcode scanner for USB connection	Z751A	Z751A
Barcode printer		
Barcode and label printer including soft-		
ware, for USB connection to the PC or test		
instrument SECUTEST BASE(10)	Z721D	Z721D
Label set for Z721D barcode and label		
printer (quantity x width: 3 x 24, 1 x 18,	77000	77000
1 x 9 mm, length: 8 m each)	Z722D	Z722D
Label set for Z721D barcode and label		
printer (qty. x width: 5 x 18 mm, 8 m long	77005	77005
each)	Z722E	Z722E
Thermal printer		
Thermal printer for printing out test re- ports; incl. manual on CD, lithium battery,		
power supply adapter, mains cable, USB		
cable, 1 role of thermal paper	Z721S	Z721S
Thermo paper for Z721S; 10 roll of thermal	21210	27210
naper VI 12/50mm 30 m x 112 mm coat-		77000
paper, Ø 12/50mm, 30 m x 112 mm, coat- ing outside	Z722S	Z722S

^D data sheet available

¹ only with SECUTEST PRO (Feature I01) or SECULIFE ST BASE

For additional information regarding accessories please refer to

- Measuring Instruments and Testers catalog
- www.gossenmetrawatt.com

Edited in Germany • Subject to change without notice • A PDF version is available on the Internet



GMC-I Messtechnik GmbH Südwestpark 15 90449 Nürnberg, Germany Phone +49 911 8602-111 Fax: +49 911 8602-777 e-mail: info@gossenmetrawatt.com www.gossenmetrawatt.com